


Zirtooth® MultiLuster

Aesthetic & Strength

- Natural gradation of shades
- Excellent strength to withstand margin chipping and fracture
- Gradation block with the same strength from cervical to incisal

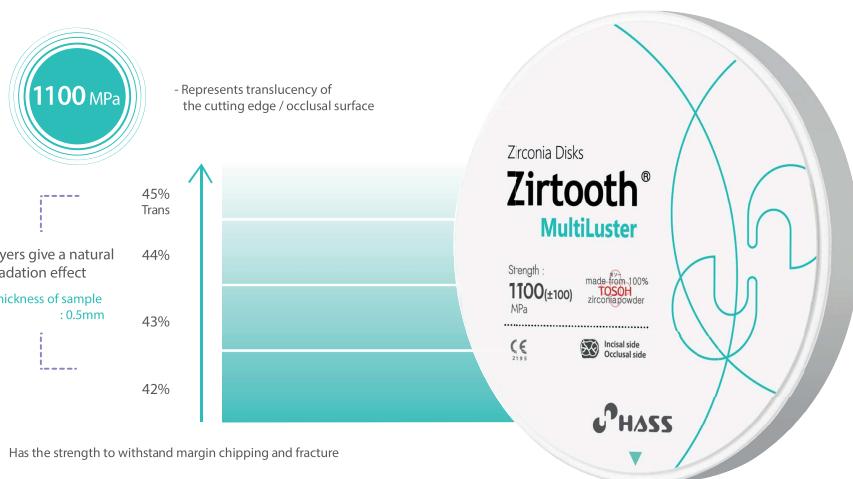


Clinical Case



Zirtooth MultiLuster A2 - Shade A2(24-25), A3(26), A3.5(27)
* after color characterization

MultiLuster

- Translucency ●●●○○
- Strength ●●●●●
- Aesthetic ●●●●○

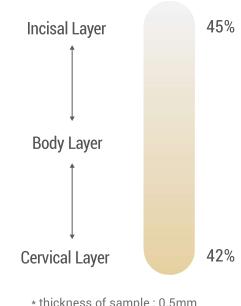

Indication

- Anterior Crown
- Posterior Crown
- 3-Unit Bridge
- Long-span Bridge



Sintering Schedule

Step	1	2	3
Temperature	Room temperature ~ 1000°C	1000°C~ 1530°C	1530°C ~ Room temperature
Heating Rate	Max 9°C	Max 3.5°C	Max 8°C
Holding Time	10 min	2 hours	-

Size

Model No.	Dimensions	Model No. * Zirkozahn	Dimensions
O-9812	ø98 x 12T	Z-95(12)	ø95 x 12T
O-9814	ø98 x 14T	Z-95(14)	ø95 x 14T
O-9816	ø98 x 16T	Z-95(16)	ø95 x 16T
O-9818	ø98 x 18T	Z-95(18)	ø95 x 18T
O-9820	ø98 x 20T	Z-95(20)	ø95 x 20T
O-9822	ø98 x 22T	Z-95(22)	ø95 x 22T
O-9825	ø98 x 25T	Z-95(25)	ø95 x 25T

Available Shades

A1, A2, A3, A3.5, A4

B1, B2, B3, B4

C1, C2, C3, C4

D2, D3, D4